



DermaPrep.™

PRODUCT INFORMATION AND INSTRUCTIONS FOR USE

BACKGROUND

The *DermaPrep*™ is made of bi-layer (soft and abrasive) sponge with an imbedded irrigation system. After opening the wrapping the *DermaPrep*™ is connected by the mean of an infusion set or a similar regulated connecting tubing to a sterile saline (or water) container (bag or bottle). After opening and regulating the flow valve the sponge is used to clean the affected tissues providing the desired quantity of fluids. The soft foam is used for a gentle cleaning whereas the abrasive side is used for a thorough cleaning of foreign materials. The sponge is squeezed after each use into the bin and thereafter refilled with new, clean and sterile cleansing liquid. The amount and nature of the cleaning liquids is easily controlled. The amount of necessary liquid is minimal and the entire volume is used effectively to clean the skin surface without any loss and dripping on the sides.

PACKAGING

- The *DermaPrep*™ is packaged in a pouch and is disinfected.
- The pouch contents is clean and disinfected unless the package is opened or damaged.
- The *DermaPrep*™ is a single use device. Do not re-use.

INDICATION

The *DermaPrep*™ is intended for use for cleansing and debridement of ski and wounds.

INSTRUCTIONS FOR USE

1. Open the package and take out the *DermaPrep*™
2. Remove the cap from the female luer lock.
3. Connect the female luer lock to I.V. line
4. Connect I.V. line to a source of sterile water or saline using I.V. giving set or extension line. .
5. Open the clamp and allow the fluid to flow into the *DermaPrep*™.
6. The wet *DermaPrep*™ is used to rub and clean the skin. Use the soft side first. For more vigorously cleaning use the abrasive side.
7. Control the flow rate of the fluid using the roller clamp.
8. Squeeze the *DermaPrep*™ into a bin in order to remove the dirt from the surface

Storage condition:

Store in cool and dry environment 0-50° C.
Relative humidity up to 80%
Keep away from direct sunlight

PRECAUTIONS AND WARNINGS

Federal (US) law restricts this device to sale by or on the order of a physician.

Warning:

Re-use of this device may cause serious deterioration in state of the patient health.

Manufacturer: 4Med Ltd.7 Shikma St. Sapirim Industrial Park. P.O. Box 1028 Sderot 8100101 Israel.
EC Authorized representative: Ozat EU, Stuttgartsrasse 15-17 Dortmund 44143, Germany

Rev f

4Med Ltd. 7, Shikma St. Sapirim Industrial Park, P.O. Box 1028 Sderot 8700101 Israel
Tel:+972 (0)8-6805223; FAX: +972(0)8 6805221
Website: www.4med.co.il e-mail:info@4med.co.il